



Giardia-Strip


Rapid diagnostic test for *in vitro* detection of *Giardia* in stool specimens

- Easy to use
- Easy to read
- Specific
- Sensitive
- Reliable
- Rapid (15')



www.corisbio.com

Giardia-Strip

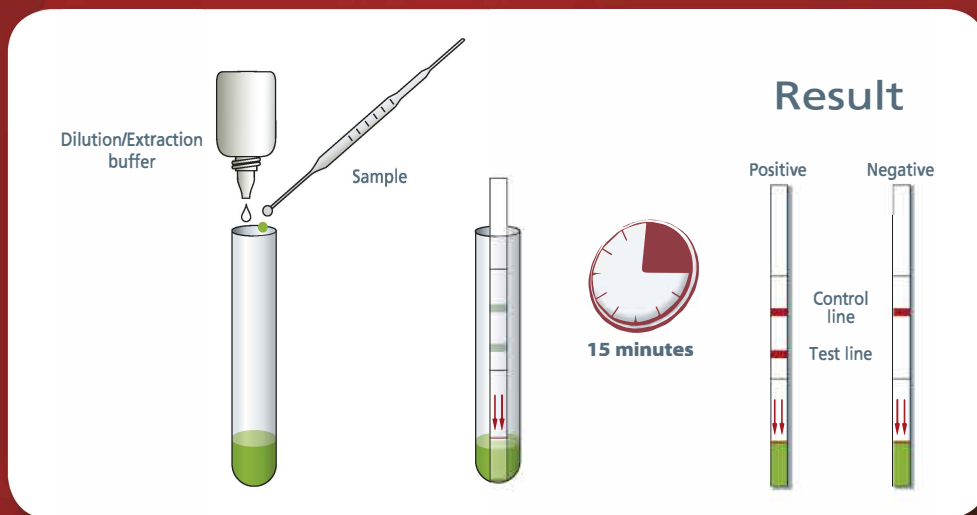

Performance compared to

	ICT competitor (n=100) Spain (Saragosse)	Microscopy (n=382) France (Lyon)
Specificity	100% (52/52)	97.8% (309/316)
Sensitivity	89.6% (43/48)	96.3% (77/80)
PPV	100% (43/43)	91.7% (77/84)
NPV	91.2% (52/57)	99% (309/312)
Accuracy	95% (95/100)	97.5% (386/396)

For more technical information

client.care@corisbio.com


Ordering Information

Product Name	Description	Storage	Code
Giardia-Strip	25 tests, buffer	4°C to 30°C	C-1013
Giardia K-SeT	20 cassettes, buffer	4°C to 30°C	K-1213
Giardia K-SeT, with collection set	20 cassettes, 20 faecal sampling systems	4°C to 30°C	K-1513

All diagnostics available in Coris BioConcept catalogue :

Adenovirus, Adenovirus (Respiratory), Adenovirus 40 and 41, *Clostridium difficile*, Bovine Coronavirus, *Cryptosporidium*, *Escherichia coli* O157, *Escherichia coli* K99, Giardia, *Helicobacter pylori*, Influenza A and B viruses, *Legionella pneumophila*, Respiratory Syncytial Virus, Rotavirus, Mefloquine, Proguanil, *Leishmania spp.*, *Trypanosoma cruzi*, *P. aeruginosa* efflux.

Distributed by:

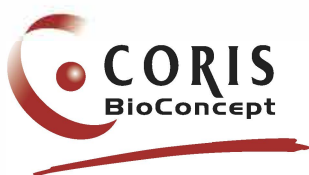

Science Park CREALYS
Rue Jean Sonet 4A - 5032 Gembloux - Belgium
Ph.: +32 (0) 81 719 910 - Fx.: +32 (0)81 719 919
sales@corisbio.com - www.corisbio.com



MADE IN BELGIUM

Fo 7.5-32-13/14
January 2012