

Approved accessories for Mermaid

Breast milk collection set

beldico manufactures approved Mermaid breast pump accessories for both hospital and domestic use.

The breast shields come in different sizes to accommodate the various sizes and shapes of nipples and areolas.

All breast milk collection sets fit any beldico feeding bottles. All material used to manufacture the accessories coming in contact with breast milk are food grade, latex free and BPA / DEHP free.



Domestic use

The **beldico** breast milk collection sets for domestic use consist of a complete set with breast shield, 120 ml feeding bottle, valve, bellow and 100 cm silicone tube. These sets are approved Mermaid breast pump accessories.

The sets are offered as a non-sterile An approved trolley kit with a fixing plate single or double set. The bellow and accessories.



Hospital use

The breast milk collection sets for hospital use are offered in various conditions:

- Sterile milk collection sets for neonatal intensive care and infants with severe pathologies (ETO sterile)
- Bacteriologically clean milk collection sets for neonatal (intensive) care units and infants with severe pathologies (gamma treated)
- Non-sterile milk collection sets for general hospital/maternity use

Trolley kit

allows secure and stable transport of the tube are also available as replacement Mermaid breast pump. This trolley is intended for hospital use only.



The breast milk collection sets contain a complete set of

- Breast shield (22, 24, 27, 30 or 36 mm)
- Feeding bottle
- Valve
- Bellow
- 100 cm silicone tube

beldico is a leader in specific feeding devices for newborns. beldico develops and manufactures medical devices for the nutrition of babies.

LIFE STARTS WITH US





Distributed by



'esucta ApS • Palle Juul-Jensens Boulevard 82 • DK-8200 Aarhus N (Denmark)



BELDICO • Rue André Feher 3 • B-6900 Marche-en-Famenne (Belgium) T. + 32 (0)84 31 37 80 • F. + 32 (0)84 31 37 87 www.beldico.com • info@beldico.be





LIFE STARTS WITH US

% Mermaid

Breast pump

Breastfeeding is the natural way to support the growth and development of your baby. World Health Organisation (WHO) advises exclusive breastfeeding for the first 6 months as the best way to give a newborn a great start in life. beldico wants to help mothers give their baby the best nutrition possible by offering a user-friendly breast pump, the Mermaid.



Comfortable milk collection

To simulate the nature of breastfeeding and in order to express more milk in less time, the Mermaid breast pump uses a dual-phase technology.

technology consists of a stimulation phase indicates the chosen settings. and an expression phase.

breast milk.

Easy to use

The unique milk collection set is easy to install by fixing the bellow effortlessly to the external piston of the Mermaid breast pump, with no additional actions required.

As soon as the machine is operating, the pumping action starts with the gentle stimulation phase to encourage the milk let-down. Subsequently, the Mermaid breast pump smoothly switches from the stimulation to the expression phase.

Each mother can customize the pump settings to her own body's response. The Mermaid even has independent controls for each breast for the highest The continuously adjustable dual-phase comfort. The digital display clearly

The breast pump can be paused and The stimulation phase imitates the fast and restarted at the mother's convenience, gentle sucking of the baby, encouraging but will automatically shut off if paused the milk let-down. The effective expression for more than 30 minutes. At this point phase imitates the slower, more intense the filled feeding bottle(s) can easily be sucking of the baby to efficiently express unscrewed from the milk collection set for further use.



Ultimately hygienic

The revolutionary concept of the complete separation between milk collection set and breast pump quarantees ultimate hygienic conditions, as only air and milk can flow between the bellow and the mother's breast.

Driven by linear mechanical movements, the patented bellow acts as a pumping device, creating the necessary vacuum within the milk expression set.

Due to this closed system, with complete visibility at any time, the air flow and the collected breast milk cannot be Our single-use sterile and bacteriologically cannot be cross-contaminated by the cleaning before or after use. content of another breast milk collection set used with the same machine.

Easy to clean and maintain Mermaid breast pump

The unique and smooth design of the Mermaid breast pump ensures that the surface is very easy to clean.

The usual hospital cleaning methods for aeneral electrical / electronic equipment are sufficient to keep the external parts of the breast pump clean.

No special maintenance of the electrical equipment is required.

Breast milk collection sets

contaminated by any part of the clean breast milk collection sets are equipment. Besides, the equipment disposable and do not require any





Quiet

Due to the patented dual-motor breast The Mermaid breast pump can be used pump technology, the Mermaid works for both single and double pumping. at low noise level. There are no vibrating Simultaneous pumping of both breasts parts that could cause unpleasant reduces the total expression time. vibrations or disturbing noise.

Compact and portable

The Mermaid breast pump has a technology very efficient. compact, lightweight design that can be attached to a trolley for safe and secure transport in a hospital environment.

With the embedded dual bottle holders, while different sizes of breast shields the Mermaid holds the beldico feeding (22, 24, 27, 30 or 36 mm) will provide a bottles upright on top of the pump, good fit for greater comfort and better reducing its space requirement.

Convenient and efficient

There is no loss of vacuum in case of dual pumping compared to single pumping, making the Mermaid pumping

In addition, the Mermaid breast pump is very convenient due to the independent setting possibilities for each breast, milk flow.

Product code	Description
	Mermaid breast pump unit with power cable, Instructions for use (IFU) and Quick Guide
5101484	Trolley with fixing plate for the Mermaid breast pump unit with ABS foot, swivelling wheels, cable hooks and adjustable height